

# Vit-I-escence<sup>®</sup>

for your **ALIGNER ATTACHMENTS**

from **ULTRADENT<sup>®</sup>**  
PRODUCTS, INC.

**SAVE TIME  
& MONEY!**

## TRANS ICE SYRINGE

Give your patients the best in  
colour match attachments!

- ~ Esthetic restorative material
- ~ A highly polishable and extremely durable micro-hybrid resin that blends with enamel
- ~ Superior handling characteristics
- ~ 2.5g syringe, shelf life 60 months



Makes cleaning up flash and removing attachment easy.

**Save 10% ON YOUR FIRST ORDER!**

Dentistry and photography courtesy  
of Dr. J. Bozek & Dre D. Larose



#745479 Trans Ice Syringe

**Orthodontic  
Essentials<sup>™</sup>**

1-866-388-1851 | WWW.ORTHODONTICESSENTIALS.COM



- 1) Clean with fluoride free pumice such as PURE - Ultrapro TX Prophy paste Medium (#818325)
- 2) Etch for 15 seconds with Opal Etch
- 3) Prime with Opal Seal and air thin



- 4) Apply Trans-ice Vit-I-escence restorative material to aligner tray and place or store in a black box for later placement

**Hint: Use a beaver tail instrument to push all four sides of the attachment against the tooth for better adaption and shape**

- 5) Cure 3 sec. with valo in plasma mode, remove tray and cure an additional 3 sec.



- 6) The final result is an almost invisible attachment that blends with shade of natural enamel.



- 7) Trans Ice has a fluorescent property for easy removal of flash and for checking voids in the tray