

NEW!

TRANSPORT™

The Handheld, Disposable Impression Syringe

from
CLINICIAN'S CHOICE
DENTAL PRODUCTS INC.

TRANSPORT is the ultimate in impressing convenience. A handheld, disposable impression syringe, TRANSPORT provides more control intra-orally compared to bulky impression guns and cartridges, and saves you money on costly mixing tips and wasted impression material!

Simple

- Load the TRANSPORT syringe directly from the impression cartridge
- Insert the TRANSPORT plunger
- Align the mixing tip and barrel by sliding the tip into place and snap the mixing tab
- Extrude the wash material around the prep

Better, Easier Control

Everyone knows handheld impression syringes provide more control, but until now, these syringes have proved to be cumbersome, costly, and time-consuming to clean.

New TRANSPORT is lightweight, fits snugly in your hand, and has a rotatable, 1.5mm angled tip to give you complete control in all hard-to-reach areas such as the distals of second molars. Compare this to maneuvering a bulky impression gun intra-orally. And, since TRANSPORT is disposable, there's no cleanup.

"All-In-One-Syringe" Saves You Money

Essentially, TRANSPORT is an "All-In-One-Syringe" giving you the syringe, mixing tip, and intra-oral placement tip for a total of just 89¢. Compare that to conventional mixing tips that together can cost as much as \$1.50 – and that's just for the tips. The cost of a typical curved disposable handheld syringe is around 50cents. TRANSPORT gives you both for 89¢ – less than half the cost!

In addition, the TRANSPORT's short, tapered mixing canula ensures you waste less impression material compared to larger conventional mixing tips which can trap up to 50% more material. Add it all up, and you save between \$1.00 and \$1.50 per impression!

Chair Side Efficiency

For unparalleled efficiency, TRANSPORT syringes can be quickly preloaded at the beginning of each day, or just before each procedure as needed, and are then ready to use chair side. A full TRANSPORT syringe of material is enough for 2 preps, and a half-filled TRANSPORT syringe is sufficient to record one crown prep. A 50ml cartridge of impression material should half-fill 42-44 TRANSPORT syringes or fully fill about 28 TRANSPORT syringes.

Instructions For Use



#1 Ensure top slide is open to let air escape (inset). Place Transport syringe barrels against cartridge and squeeze gun gently. If using a new cartridge, bleed a pea-size amount of material to ensure even fill (base/catalyst).



#2 For 1 prep, fill halfway. For 2 preps, fill the syringe, leaving enough space for the plunger ends to be inserted without premature material extrusion at the Transport mixing section (a little practice is necessary).



Gently insert the plunger to seal the end of the barrels of the Transport syringe.



#3 Slide the mixing tip section all the way across until the tab snaps off.

Depress the plunger to mix and then syringe around the preparation, keeping the tip submerged in the wash material to avoid air entrapment.



To Pre-Load For Later Use

Follow steps 1 and 2 above. Slide mixing tip section partway until you hear/feel a click and the O-rings are completely sealed. It is important that you do not snap the tab off during this step. The Transport syringe is now ready for storage until needed.



EXCLUSIVE TO:

CLINICAL RESEARCH DENTAL
SUPPLIES & SERVICES INC.



116100 Transport Syringe 100-pack

1-800-265-3444

www.clinicalresearchdental.com