



How To Clean Your VALO Ortho

Routine Maintenance and Cleaning

After each use, moisten a gauze or soft cloth with an anti-microbial surface disinfectant and wipe surface and lens. In the event that dental resin adheres to the VALO lens, use a non-diamond dental instrument to carefully remove the resin.

WARNING:

DO NOT autoclave
DO NOT immerse in any kind of ultrasonic bath or in any liquids
DO NOT wipe down the VALO curing light with caustic or abrasive cleaners or with peroxide-based cleaners. These will remove the Teflon coating!

Acceptable Cleaners:

- Cavicide products
- Isopropyl alcohol-based cleaners
- Ethyl alcohol-based cleaners
- Lysol disinfectant
- Other non-bleach and non-abrasive disinfectants or cleansers

NEVER Use:

- Peroxide-based cleaners (e.g. – *optim 33TB wipes*)
- Formula 409 based cleaners
- Bleach-based cleaners (e.g. – *Clorox, Sterilox*)
- Abrasive Cleansers (e.g. – *Comet Cleanser*)
- Acetone-based cleaners (e.g. – *nail polish remover, Goo-Gone*)

Warranty: 2 years

Warranty applies solely to original purchaser and is not transferable. There are no user service components of the VALO system. Tampering with VALO will void its warranty.